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Tandheelkundig lepelmateriaal kortstondig

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**Intended use**  
Matériau splinting material short-term

**Extended purpose**  
Thermoforming blank for the production of bleaching splints.

**Indication for use**

FOLIFLEX BLEACH was especially designed as a splint material for home-bleaching. Also after the thermo-forming process the special formulation of the 1.0 mm strong EVA-material offers a consistent soft and stable splint. The usage for several bleaching processes is possible without any problems.

**Contraindication**

In case of allergies to any of the ingredients do not use FOLIFLEX BLEACH. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.

**Target patient group**

Patients who need dental care.

**Intended users**

Trained staff in dental field

**Characteristics**

Flexible, soft-elastic thermoforming foil made of ethyl-vinyl-acetat (EVA)
∅ 120 mm / 125 mm
□ 125 mm x 125 mm

**Material thickness**  
1,0 mm

**Color**

Transparent

**Preparative measures**

Before thermo-forming, the blank a space maintainer as a reservoir for the bleaching material has to be applied on the labial areas of the teeth. For this exist two possibilities:

a) With a light-curing easy flowing but thixotropic resin material the areas at the plaster model which are supposed to be bleached are consistently covered with a brush and afterwards get cured.

or

b) The whole model is covered with the thermo-forming blank FOLILEN 0.8 mm. Then, the part of the model to be bleached (incl. a slightly overlapping of the cutting edges) is cut out of the blank by means of a scissors and put back onto the model again.

This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before

**IMPORTANT:** In order to avoid any contact between the mucosa and the bleaching material the space maintainer should end approx. 1 mm in front of the gingival margin.

**Processing instruction**

Insert the FOLIFLEX BLEACH blank in the thermo-forming unit and heat it. The right plastification is achieved when the blank sags for approx. 10–15 mm.

After having achieved the right plastification drag the blank directly over the model and form it with vacuum or pressure. **IMPORTANT:** In order to realise a good and stress free splint the pressure/vacuum has to be adhered for approx. 5 min. until the complete heat exchange of the blank. Before thermoforming plaster models should be watered and insulated.

**Processing instruction**

The detaching of the splint from the cooled down thermo-forming blank is effected by means of a scissors or a hot scalpel. Due to the good elasticity of the splint material the marginal limit of the blank can outreach the gingival margin. This extends the vent edge of the splint and therefore guarantees the safe position of the bleaching material in the reservoir. In order to avoid irritations of the gingiva and the cheek the cut edges of the thermoforming splint should carefully be rounded off with a rotating instrument and smoothed.

**Disinfection and reuse**

A disinfection with all commercial materials is possible. FOLIFLEX BLEACH used as a bleaching splint, can usually be reused maximum 4 times for 20 minutes each time.

**Note**

- While working out on the material, it is recommended to use an exhaust system and protective glass or goggles as well as a face mask.

**Serious incidents**

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

**Storage conditions**

At room temperature, i. e. 18–28 °C.

**Lot number / Expiry date**

The lot number and the expiry date are indicated on each packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

**Disposal**

Disposal of contents/the packaging in accordance with local/regional/national/international regulations.

using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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